

# BENZOMIL<sup>\*</sup> PLUS

## The High Level Disinfectant & Fogging Solution

### Benefits:

- Broad antimicrobial action
- Excellent synergistic effect
- High organic tolerance
- Ready to use, no activation required
- Excellent material compatibility

### Composition:

Glutaraldehyde ..... 2% w/v  
Benzalkonium chloride solution IP ..... 5% w/v  
Corrosion Inhibitors

### Area of usage:

**Pack Size: 1 Litre**

Surface disinfection in Hospitals, OT's, ICU's, Polyclinics, Dental Clinics, OPDs, Blood Banks, Clinical/Diagnostic laboratories, flexible endoscopes and Surgical equipment.

### Direction of Use:

#### High Level Disinfection:

- Prepare 20%v/v of Benzomil Plus Solution and dip the clean surgical equipment & endoscope for 15min in prepared solution. Rinse with water.

#### Sterilization:

- Immerse clean surgical equipment & endoscopes in fresh Benzomil Plus solution for 5 hrs. Finally rinse with water.

#### Fumigation/Fogging:

- For aerial fumigation or fogging of critical locations such as OT's, QA/QC laboratories, Production areas, and clean room/Sterile areas in industrial settings; use 5%v/v. Follow fogging or fumigation, allow 1 hr for contact.

#### Surface Disinfectant:

- Use 5% v/v solution prepared in DW for everyday cleaning and surface disinfection in OT's, QA/QC labs, R&D labs, Production areas, and clean room /sterile areas in commercial settings.



[www.millaboratories.in](http://www.millaboratories.in)



022 -26143002 /26138700



[info@millaboratories.in](mailto:info@millaboratories.in)